## IT ISOMAR® 20 Flaconcini Igiene Quotidiana

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## EN ISOMAR® 20 Daily Hygiene Vials

Isotonic and sterile seawater solution for daily nasal and ocular hygiene of children and adults. The sea water is drawn from Cinque Terre (Liguria, Italy) 3.5 miles from the coast and at a depth of 6 metres.

**INDICATIONS FOR THE DAILY NASAL HYGIENE:** in case of cold, sinusitis, rhinitis, allergies in general also caused by pollen for people working outdoors, above all in presence of powder. As a solvent of mucus and blood residues of the mucous membrane of the nose, for the hygiene and cleaning of the mouth and eyes of children, for the hygiene and cleaning of nipples on breastfeeding women. In the adults, it allows a better draining of the mucous nasal membrane. It is indispensable for people who underwent nasal operations and for who snore.

**INDICATIONS FOR EYE WASHING:** suitable in case of dryness and non pathological troubles such as eye-burn, reddening due to external factors such as: wind and solar exposition, powder, pollen, smoke, saltiness, dry environments (air conditioning, dry warm...), prolonged stay in front of screens and use of contact lenses.

**ISOMAR® Daily Hygiene Vials** are recommended for use in aerosol therapy to humidify the upper airways.

It is recommended for use in children over 6 months, adults and pregnant women.

**USE:** open the sealed up vial turning the cap. Put the mono dose vial in the nostrils of the baby, who is laid down. Press the vial till to have the desired quantity (See picture 1). For the child who use **ISOMAR® Daily Hygiene Vials** by himself and for the adults, the vial must be put in the nostrils, pressing it head up breathing in and closing the other nostril with the finger (See picture 2).

**FREQUENCY OF USE:** up to a maximum of 3 times a day and as needed.

**USE AND PRECAUTIONS FOR EYE WASHING:** instill one or more drops in the lachrymal sac, according to one's necessity, even frequently during the day. The top of the vial must not come into contact with the surface of the eyelid or the eye (See picture 3).

**FOR AEROSOL THERAPY:** place the liquid into the aerosol therapy device, following the manufacturers' instructions.

**CONTRAINDICATIONS:** do not use **ISOMAR® Daily Hygiene Vials** in infants younger than 6 months old. Do not use in case of epistaxis.

**WARNING:** sterile solution not suitable for injection. Keep the vial out of the reach of children. Do not use the same single-dose vial both for nasal hygiene and for applying to the eyes; for aerosol therapy, use a dedicated vial. Do not mix with other solutions. After opening, the vial must be used immediately and disposed of after use. Do not dispose of empty vials in the environment. Do not use after the expiry date. If the product is not intact, its functional properties may be altered. Store in a dry place, away from heat sources at a maximum temperature of 25°C. Do not use concomitantly with other topical treatments. Do not use with contact lenses. **ISOMAR®** is proposed also in 100 ml Spray with nebulizing pump for nasal hygiene with alternate nebulizer, in the box, for the hygiene of the ear. During aerosol therapy, we recommend alternating the use of **ISOMAR® Daily Hygiene Vials** with pharmacological treatments.

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