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## EN ISOMAR® Decongestant Spray no gas

Seawater Hypertonic Solution and Eucalyptus Essential Oil.

ISOMAR® Decongestant Spray no gas is a hypertonic solution based on seawater (Mediterranean, Cinque Terre, Liguria Italy) in its pure state designed to have a natural decongestant action of the nasal mucous membranes, that promotes rapid relief in the event of a blocked nose. Hypertonic seawater frees the nose from excess mucus by osmosis, does not induce habituation, does not damage the mucosa. The essential oil of Eucalyptus and the aroma of mint give a feeling of balsamic freshness.

**INDICATIONS:** ISOMAR® Decongestant Spray no gas is indicated for the following uses: to fluidify and improve the drainage of mucus and reduce edema in the upper airways; to decongest the nasal passages from stagnant mucus in cases of cold and allergic rhinitis; to soothe the symptoms of sinusitis and rhinosinusitis; to thoroughly cleanse the nasal cavities by removing dust and smog.

ISOMAR® Decongestant Spray no gas is contained in a 30ml bottle.

**COMPOSITION:** Hypertonic seawater (99,6%), Eucalyptus Globulus leaf oil, Glycerin, Aqua, Mentha Piperita Leaf Extract.

**CONTRAINDICATIONS:** do not use in children under 2 years of age. Do not use in case of hypersensitivity to one or more of the components.

**HOW TO USE:** for proper use, remove the cap and, holding both the head and the bottle vertically, insert the opening of the dispensing spout inside the nostril, without sinking excessively into the nasal cavity. Press with two fingers on the base of the dispenser and inhale at the same time, as the solution comes out. Repeat the operation with the other nostril. Rinse the dispensing spout with water before replacing the cap. Procedure: 1-2 sprays per nostril up to a maximum of 3 times a day. Allow the solution to act for a few seconds before blowing your nose. Do not exceed the recommended dose. Do not exceed two weeks of treatment.

**WARNINGS:** does not contain propellant gas. Do not use in conjunction with other topical treatments. Do not exceed the recommended dose. Keep out of the sight and reach of children. Use the product exclusively for its intended use. It can be used on children from 2 years of age and must be applied by an adult. Failure to check the integrity of the product could alter its functional characteristics. Do not use the product after the expiry date. Do not use if package is damaged. Store at a temperature between 5°C and 25°C (41°F - 77°F), away from light and heat sources. Do not dispose of in the environment after use. In the event of a serious accident occurring in relation to the medical device, immediately report it to the Competent Authority of the Member State in which the accident occurred.

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