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EN ISOMAR® Decongestant Spray with Hyaluronic Acid

Hypertonic seawater solution from Cinque Terre

Isomar Decongestant Spray with Hyaluronic Acid has a dual action for the well-being of the respiratory tract:

1 **Pure seawater** is a natural decongestant and fluidifying agent that rapidly helps free blocked nose. It improves drainage of secretions, keeping the nasal passages clean and facilitating breathing.

2 **Hyaluronic acid** keeps the mucous membranes soft and protects cells from dehydration, reducing the risk of dryness and micro injuries in the nasal cavities that may occur in case of severe colds, allergic rhinitis or sinusitis.

COMPOSITION: Hypertonic seawater (natural seawater salt concentration of around 3%), sodium hyaluronate. Inert propellant gas: nitrogen. Preservative and dye free.

RECOMMENDED USE: Isomar Decongestant Spray with Hyaluronic Acid is recommended for fast relief from congested nose in the event of colds, bronchitis, sinusitis, allergies in general (including pollen). The product is suitable for use in adults and children over 12 months. The product is non-addictive, does not damage the nasal mucosa, it does not produce overdose and can be used even during pregnancy. Hyaluronic acid makes it suitable for frequent use.

DIRECTIONS FOR USE: Gently introduce the nozzle into the nostril and spray 2-3 times per nostril up to a maximum of three times a day. Leave for a few seconds before blowing the nose. After using Isomar Decongestant Spray with Hyaluronic Acid, remove, wash and dry the nozzle. If the problem persists, consult your doctor or a specialist. Do not use for more than two weeks. Do not exceed the recommended dose. Isomar Spray can be used both vertically and horizontally to make nasal washing for adults and children even easier and more thorough. VERTICAL: the vertical use of the spray allows for the better reaching of all the grooves within the nasal septum, for quicker and more complete cleansing. HORIZONTAL: in children over the age of 12 months, apply the product in a horizontal position.

CONTRAINDICATIONS: Do not use Isomar Decongestant Spray with Hyaluronic Acid in children younger than 12 months. Do not use in case of epistaxis.

WARNINGS: Pressurized container: May burst if heated. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn, even after use. Protect from sunlight. Do not expose to temperatures exceeding 50° C/122° F. Do not use concomitantly with other topical treatments. Keep out of reach of children. For intended use only. The product is suitable for use in children over 12 months and must be applied by an adult. Failure to ensure the integrity of the product may alter its functional characteristics. Do not use after the expiry date. Store at a temperature between 5° C and 25° C (41° F - 77° F). In some cases the product may be slightly irritating.

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